

INHEALTH
TECHNOLOGIES®

PRODUCT CATALOGUE



TOTAL

Solution

CONTENTS

Pre-Operative Education	2
"What to Expect" Brochure	2
Educational Videos	2
Surgical Solutions	3
Tracheoesophageal Puncture Set (Sterile)	3
Indwelling Voice Protheses (Sterile) Adjustable	3
Bi-Flanged Fistula Protheses (Sterile)	3
Post-Operative Care	4
Laryngectomy Tubes	4
StomaSoft Cleaning Swabs	4
Post-Operative Care Kits (For Total Laryngectomy)	5
Next Steps	6
Voice Protheses (Clinician Placed)	6
Special Order Voice Protheses (Clinician Placed)	8
Voice Protheses (Patient Changeable)	10
Voice Prosthesis Accessories	11
Heat & Moisture Exchange (HME)	13
Adhesive Housings	14
Tracheostoma Attachments	14
And Beyond	15
Skin Care	15
Stoma Protection and Accessories	15
SEVERN DIRECT Home Delivery Prescription Service	16
Professional Products	17
Voice Prosthesis Management	18
Oto-Rhino Foreign Body Remover	20
Nasal Septal Perforation Protheses	22

InHealth Technologies® offers a TOTAL Solution for your total laryngectomy from pre- to post-op and beyond

TOTAL Solution serves patients at all phases of voice restoration, from products that deliver solutions for pre- and post-operative care and beyond. This continuum of care is enhanced by InHealth Technologies' customer-focused team, including leading clinical specialists, experienced patient consultants and dedicated customer service members.



Cover Image:

Left: Guiermo
Year of TL: 2021
Wearing Accufit housing
and Day&Night HME

Right: Ksenia
Year of TL: 2019
Wearing Accufit housing
and Day&Night HME

TOTAL Solution

PHASE

1

Pre-Operative Education

- "What to Expect" Brochure
- Educational Videos

PHASE

2

Surgical Solutions

- Voice Prosthesis Placement Surgical Kits
- Sterile Option Voice Prostheses
- Fistula Prostheses

PHASE

3

Post-Operative Care

- Post-Operative Care Kits
- Laryngectomy Tubes
- Electrolarynx
- "Life After Laryngectomy" Brochure

PHASE

4

Next Steps

- Next Steps Kit
- Voice Prostheses
- HMEs
- Adhesive Housings
- Educational Programs

PHASE

5

And Beyond


- Hands-Free Speech Valve
- Skin Care

● PRE-OPERATIVE EDUCATION

"What to Expect" Brochure

Available online and free to download at:

<https://inhealth.com/educational-materials>

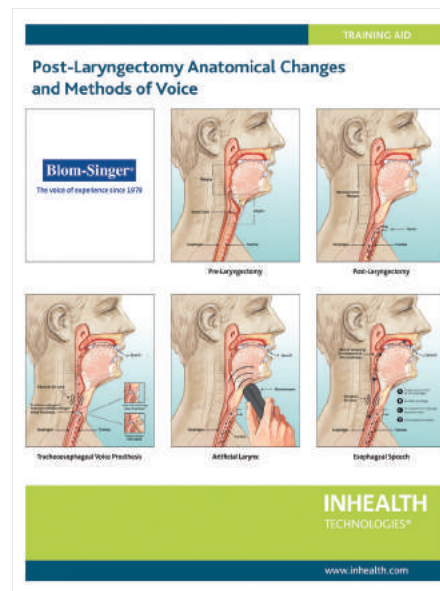

Educational Videos

Available at InHealth Technologies®

<https://inhealth.com/videos>



● SURGICAL SOLUTIONS

Blom-Singer® Tracheoesophageal Puncture Set (Sterile)

Designed for secondary placement of a tracheoesophageal prosthesis. Includes a disposable 16 gauge curved introducer needle with a 14 gauge Teflon® sheath, a 26" (66cm) silicone catheter with a tapered reinforced tip attached to a 24" (61cm) Teflon-coated stainless steel leader cable, and a fenestrated surgical drape with adhesive around the aperture.

Ref	Description
BE 6055	Tracheoesophageal Puncture Set (Sterile)

Blom-Singer® Classic™ Indwelling Voice Prostheses (Sterile)


Designed for primary TE puncture and placement procedures. Includes insertion and cleaning accessories. Clinician placed.

Length	16 Fr	20 Fr	Length	16 Fr	20 Fr
8mm	IN 1608-SO	IN 2008-SO	12mm	IN 1612-SO	IN 2012-SO
10mm	IN 1610-SO	IN 2010-SO	14mm	IN 1614-SO	IN 2014-SO

Blom-Singer® Adjustable Bi-Flanged Fistula Prostheses (Sterile)

Designed by Eric D. Blom, Ph.D to control leakage of saliva, food, and drink around a hypopharyngeal fistula. The prosthesis is adjustable in length and comes in two flange sizes, 25mm and 38mm, ensuring a proper fit.

Ref	Description
FM 1003	25mm (Small) Adjustable Bi-Flanged Fistula Prosthesis, 1 ea
FM 1004	38mm (Medium) Adjustable Bi-Flanged Fistula Prosthesis, 1 ea

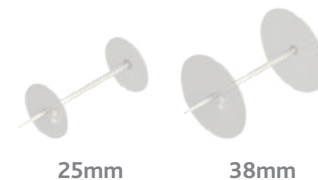

● POST-OPERATIVE CARE

Blom-Singer® StomaSoft® Laryngectomy Tubes

Blom-Singer StomaSoft Laryngectomy Tubes are flexible, soft, and compatible with HME and other accessories from InHealth Technologies®. They feature reinforced strap holders and are available with or without fenestrations.

Size/Length	I.D./O.D.	Non-Fenestrated	Fenestrated	Rx
8/36	9.5mm/12.0mm	BE 6698	BE 6698F	■
8/55	9.5mm/12.0mm	BE 6699	BE 6699F	■
9/36	10.5mm/13.5mm	BE 6700	BE 6700F	■
9/55	10.5mm/13.5mm	BE 6701	BE 6701F	■
10/36	12.0mm/15.0mm	BE 6702	BE 6702F	■
10/55	12.0mm/15.0mm	BE 6703	BE 6703F	■
12/36	13.5mm/17.0mm	BE 6704	BE 6704F	■
12/55	13.5mm/17.0mm	BE 6705	BE 6705F	■



Blom-Singer® Laryngectomy Tubes (Sterile)

Blom-Singer Laryngectomy Tubes (Sterile) are durable, lightweight silicone tubes that are used to keep the tracheostoma open after laryngectomy. The tubes do not hold an HME.

Size/Length	I.D./O.D.	Non-Fenestrated (Sterile)	Rx
8/36	9.5mm/12.0mm	BE 6398	■
8/55	9.5mm/12.0mm	BE 6399	■
9/36	10.5mm/13.5mm	BE 6400	■
9/55	10.5mm/13.5mm	BE 6401	■
10/36	12.0mm/15.0mm	BE 6402	■
10/55	12.0mm/15.0mm	BE 6403	■
12/36	13.5mm/17.0mm	BE 6404	■
12/55	13.5mm/17.0mm	BE 6405	■




Blom-Singer® Post-Operative Care Kits for Total Laryngectomy

INHEALTH
TECHNOLOGIES®

NEW & IMPROVED KITS INCLUDE:

- 2 ea. StomaSoft® Laryngectomy Tubes
- 24hr Day&Night HME® Cartridges in ClassFlow® or EasyFlow®
- 7 ea. Tracheostomy Collars
- HydroFit Adhesive Housings, 6 pack

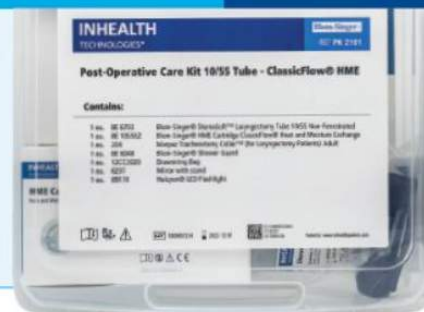

REF	Description
PK 2110	Post-Operative Care Kit 9/36 Tube - ClassicFlow® HME includes: x2 StomaSoft 9/36 tubes, x1 box of Day&Night HME ClassicFlow cassettes, x7 Tracheostomy collars x1 pack of 6 HydroFit Adhesive Housings Also included are: Blom-Singer Shower Guard, Mirror with stand, Halcyon LED Torch and Drawstring Bag
PK 2120	Post-Operative Care Kit 9/36 Tube - EasyFlow® HME includes: x2 StomaSoft 9/36 tubes x1 box of Day&Night HME EasyFlow cassettes x7 Tracheostomy collars x1 pack of 6 HydroFit Adhesive Housings Also included are: Blom-Singer Shower Guard, Mirror with stand, Halcyon LED Torch and Drawstring Bag

● NEXT STEPS

VOICE PROSTHESES (CLINICIAN PLACED)

Blom-Singer® Classic™ Indwelling Voice Prostheses (Sterile)

Includes sterile prostheses for primary TE puncture and placement procedures. Includes insertion and cleaning accessories. Clinician placed.

Length	16 Fr	20 Fr	Rx	Length	16 Fr	20 Fr	Rx
8mm	IN 1608-SO	IN 2008-SO	■	12mm	IN 1612-SO	IN 2012-SO	■
10mm	IN 1610-SO	IN 2010-SO	■	14mm	IN 1614-SO	IN 2014-SO	■



Classic (Sterile)

Blom-Singer® Classic™ Indwelling Voice Prostheses

Includes nonsterile voice prosthesis for secondary or replacement TEP procedures, with insertion/cleaning accessories. Clinician placed.

Length	16 Fr	20 Fr	Rx	Length	16 Fr	20 Fr	Rx
4mm	IN 1604-NS	IN 2004-NS	■	14mm	IN 1614-NS	IN 2014-NS	■
6mm	IN 1606-NS	IN 2006-NS	■	16mm	IN 1616-NS	IN 2016-NS	■
8mm	IN 1608-NS	IN 2008-NS	■	18mm	IN 1618-NS	IN 2018-NS	■
10mm	IN 1610-NS	IN 2010-NS	■	20mm	IN 1620-NS	IN 2020-NS	■
12mm	IN 1612-NS	IN 2012-NS	■				



Classic

Blom-Singer® Dual Valve™ Indwelling Voice Prostheses

The Blom-Singer Dual Valve Indwelling Voice Prosthesis has two valves embedded with silver oxide as a material preservative. When the primary valve fails, the secondary speech valve is intended to remain viable to help prevent leakage of fluids. The Blom-Singer® Dual Valve Voice Prosthesis is available with standard or large tracheal/oesophageal flanges. Includes nonsterile voice prosthesis for secondary or replacement TEP procedures, with insertion/cleaning accessories. Clinician placed.

Length	20 Fr	20 Fr Large Flange	Rx	Length	20 Fr	20 Fr Large Flange	Rx
6mm	DV 2006-NS	DV 2006-LF	■	12mm	DV 2012-NS	DV 2012-LF	■
8mm	DV 2008-NS	DV 2008-LF	■	14mm	DV 2014-NS	DV 2014-LF	■
10mm	DV 2010-NS	DV 2010-LF	■				



Dual Valve



Dual Valve Large Flange

Blom-Singer® Advantage® Indwelling Voice Prostheses

The Blom-Singer Advantage Indwelling Voice Prosthesis is embedded with silver oxide as a material preservative.* Includes nonsterile voice prosthesis for secondary or replacement TEP procedures, with insertion/cleaning accessories.

HARD VALVE ASSEMBLY

Length	20 Fr	Rx
4mm	AD 2004-NS	■
6mm	AD 2006-NS	■
8mm	AD 2008-NS	■
10mm	AD 2010-NS	■
12mm	AD 2012-NS	■
14mm	AD 2014-NS	■

SOFT VALVE ASSEMBLY

Length	16 Fr	20 Fr	Rx
6mm	AS 1606-NS	AS 2006-NS	■
8mm	AS 1608-NS	AS 2008-NS	■
10mm	AS 1610-NS	AS 2010-NS	■
12mm	AS 1612-NS	AS 2012-NS	■
14mm	AS 1614-NS	AS 2014-NS	■



Hard Valve Assembly



Soft Valve Assembly

*Silver oxide, a known anti-microbial substance, has been incorporated into the silicone flap valve of the Blom-Singer Advantage and Dual Valve Indwelling Voice Prostheses as a material preservative.



Left: Janice
Year of TL: 2014
Wearing AccuFit
housing with
SpeakFree HME
Hands-Free Valve

Right: Mackayla
Year of TL: 2021
Wearing HydroFit
housing and
Day&Night HME

SPECIAL ORDER VOICE PROSTHESES (CLINICIAN PLACED)


—Developed by Eric D. Blom, Ph.D

Unique problems call for unique solutions. InHealth Technologies® offers several voice prostheses to address those needs.

Special Length

Available in one millimeter incremental lengths for a more precise TE puncture fit.


	5mm	7mm	9mm	Rx
16 Fr	IN 1605-SL	IN 1607-SL	IN 1609-SL	■
20 Fr	IN 2005-SL	IN 2007-SL	IN 2009-SL	■

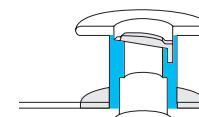

Increased Resistance

Designed for patients who experience intermittent valve opening with fluctuations in oesophageal pressures, resulting in air ingestion and/or leakage.


	6mm	8mm	10mm	12mm	Rx
16 Fr	IN 1606-IR	IN 1608-IR	IN 1610-IR	IN 1612-IR	■
20 Fr	IN 2006-IR	IN 2008-IR	IN 2010-IR	IN 2012-IR	■

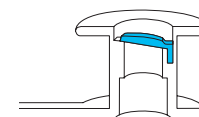

Large Oesophageal Flange


Provides a seal to help manage leakage around a voice prosthesis. Also, it enables downsizing from a wider diameter prosthesis to a smaller diameter. Effectiveness depends on an optimal prosthesis length fit that achieves a snug flange-to-flange seal against the wall surface(s).

	4mm	6mm	8mm	10mm	12mm	14mm	Rx
16 Fr	IN 1604-LEF	IN 1606-LEF	IN 1608-LEF	IN 1610-LEF	IN 1612-LEF	IN 1614-LEF	■
20 Fr	IN 2004-LEF	IN 2006-LEF	IN 2008-LEF	IN 2010-LEF	IN 2012-LEF	IN 2014-LEF	■

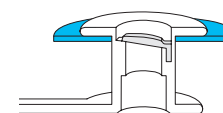

LARGE OESOPHAGEAL FLANGE (SPECIAL LENGTH)

	5mm	7mm	Rx
16 Fr	IN 1605-LESL	IN 1607-LESL	■
20 Fr	IN 2005-LESL	IN 2007-LESL	■

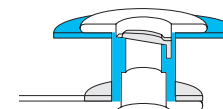

Large Oesophageal and Tracheal Flanges

Provides a double seal in patients with enlarged puncture tracts and tear-drop or irregularly shaped punctures. Effectiveness depends on an optimal prosthesis length fit that achieves a snug flange-to-flange seal against the wall surface(s).

	4mm	6mm	8mm	10mm	12mm	14mm	Rx
16 Fr	IN 1604-LF	IN 1606-LF	IN 1608-LF	IN 1610-LF	IN 1612-LF	IN 1614-LF	■
20 Fr	IN 2004-LF	IN 2006-LF	IN 2008-LF	IN 2010-LF	IN 2012-LF	IN 2014-LF	■




LARGE OESOPHAGEAL FLANGE (INCREASED RESISTANCE)

	4mm	6mm	8mm	Rx
16 Fr	IN 1604-LEIR	IN 1606-LEIR	IN 1608-LEIR	■
20 Fr	IN 2004-LEIR	IN 2006-LEIR	IN 2008-LEIR	■

TEP Occluders

The Blom-Singer® TEP (Tracheoesophageal Puncture) Occluder is a non-valved silicone device designed for placement in, and maintenance of, the TEP following total laryngectomy. Not a voice prosthesis.

	4mm	6mm	8mm	10mm	12mm	14mm	Rx
16 Fr	IN 1604-LETO	IN 1606-LETO	IN 1608-LETO	IN 1610-LETO	IN 1612-LETO	IN 1614-LETO	■
20 Fr	IN 2004-LETO	IN 2006-LETO	IN 2008-LETO	IN 2010-LETO	IN 2012-LETO	IN 2014-LETO	■

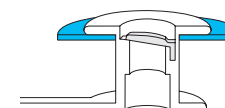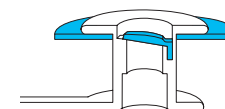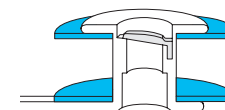

VOICE PROSTHESES (PATIENT CHANGEABLE)

Blom-Singer® Duckbill Voice Prostheses

The Blom-Singer® Duckbill is a patient changeable, slit valve prosthesis.

Length	16 Fr	Rx	Length	16 Fr	Rx
6mm	DB 16-006	■	12mm	DB 16-012	■
8mm	DB 16-008	■	14mm	BE 6001	■
10mm	BE 6000	■	18mm	BE 6002	■

Blom-Singer® Low Pressure Voice Prostheses

The Blom-Singer® Low Pressure is a patient changeable, one-way valve prosthesis.

Length	16 Fr	20 Fr	Rx	Length	16 Fr	20 Fr	Rx
6mm	BE 6009	BE 2014	■	18mm	BE 6012	BE 2026	■
8mm	LP 16-008	LP 20-008	■	22mm	BE 6013	BE 2030	■
10mm	BE 6010	BE 2018	■	25mm	BE 6014	BE 2033	■
12mm	LP 16-012	LP 20-012	■	28mm	BE 6015	—	■
14mm	BE 6011	BE 2022	■				

Blom-Singer® Special Order Low Pressure Voice Prostheses

The Blom-Singer® Special Order Low Pressure is a patient changeable, one-way valve prosthesis.

Length	16 Fr Special Increased Resistance	16 Fr Special Length	20 Fr Special Length	20 Fr Special Increased Resistance	Rx
4mm	—	SPECIAL 11-902-06	SPECIAL 11-251-06	—	■
6mm	LP 16-006-IR	—	—	LP 20-006-IR	■
8mm	LP 16-008-IR	—	—	LP 20-008-IR	■
10mm	LP 16-010-IR	—	—	LP 20-010-IR	■
12mm	LP 16-012-IR	—	—	LP 20-012-IR	■
14mm	LP 16-014-IR	—	—	LP 20-014-IR	■
16mm	—	SPECIAL 11-902-04	SPECIAL 11-251-04	—	■
18mm	LP 16-018-IR	—	—	—	■
20mm	—	SPECIAL 11-902-05	SPECIAL 11-251-05	—	■
22mm	LP 16-022-IR	—	—	—	■
25mm	LP 16-025-IR	—	—	—	■
28mm	LP 16-028-IR	—	—	—	■



VOICE PROSTHESIS ACCESSORIES

Blom-Singer® Flushing Device and Cleaning Brushes

For routine cleaning of a voice prosthesis as recommended by your clinician.

Ref	Description
IN 4050	Flushing Device, pack of 3
IN 4062*	Cleaning Brush, 16/20 Fr, Standard, 4mm–10mm, 3 ea
IN 4063*	Cleaning Brush, 16/20 Fr, Long, 12mm–20mm, 3 ea

*Fits indwelling style voice prostheses only.

*Awaiting launch on Prescription



Blom-Singer® Plug Inserts

The Blom-Singer Plug Insert is designed for temporary management of leakage through a voice prosthesis. Not compatible with patient changeable voice prostheses.

Ref	Description	Rx
IN 4069	16 Fr Plug Insert (Advantage and Classic, 6mm to 20mm)	■
IN 4070	20 Fr Plug Insert (Advantage and Classic, 6mm to 20mm)	■



Blom-Singer® HMEs

Simple choices to fit your lifestyle

Day&Night® HME



- Wear all day and all night, including bedtime
- Attaches to the entire family of InHealth baseplates and StomaSoft tubes.



Day&Night®
EasyFlow® HME and
ClassicFlow® HME

SpeakFree® HME

- Your HME and hands free speaking valve, all in one.
- Attaches to the entire family of InHealth baseplates and StomaSoft tubes.



SpeakFree®
EasyFlow® HME and
ClassicFlow® HME

Left: Joanne
Year of TL: 2015
Wearing AccuFit
housing with
SpeakFree HME
Hands-Free

Right: Meagon
Year of TL: 2016
Wearing HydroFit
housing with
Day&Night HME



HEAT & MOISTURE EXCHANGE (HME)

Research shows that long-term HME use results in less mucus¹, less coughing², and decreased shortness of breath, fatigue, sleep problems, anxiety, and depression^{1,2}.

Blom-Singer® SpeakFree® Hands-Free Valve with ClassicFlow® HME

Blom-Singer SpeakFree HME Hands-Free Valve is the first heat and moisture exchange (HME) cartridge with a single-use fully integrated adjustable hands-free valve. The SpeakFree HME is translucent and non-reflective for a natural look. The SpeakFree ClassicFlow HME is designed for increased moisture retention and resistance.

Ref	Description
BE 1090EZ	SpeakFree Hands-Free Valve with ClassicFlow HME, 30 per pack



SpeakFree
ClassicFlow

Blom-Singer® SpeakFree® Hands-Free Valve with EasyFlow® HME

Blom-Singer SpeakFree HME Hands-Free Valve is the first heat and moisture exchange (HME) cartridge with a single-use fully integrated adjustable hands-free valve. The SpeakFree HME is translucent and non-reflective for a natural look. With SpeakFree EasyFlow, the foam filter is more porous and allows you to breathe more freely to match your activity level and pulmonary needs.

Ref	Description
BE 1090EF	SpeakFree Hands-Free Valve with EasyFlow HME, 30 per pack



SpeakFree
EasyFlow

Blom-Singer® Day&Night® ClassicFlow® HME

The Blom-Singer Day&Night ClassicFlow can be worn all day and night including bedtime. The Day&Night HME is translucent and non-reflective for a natural look.

Ref	Description
BE 1091	Day&Night ClassicFlow HME Cartridge, pack of 30 (24-hour use)



Day&Night
ClassicFlow

Blom-Singer® Day&Night® EasyFlow® HME

The Blom-Singer Day&Night EasyFlow allows you to breathe more freely to match your activity level and pulmonary needs. The Day&Night HME is translucent and non-reflective for a natural look.

Ref	Description
BE 1092	Day&Night EasyFlow HME Cartridge, pack of 30 (24-hour use)



Day&Night
EasyFlow

1. Ackerstaff AH, Hilgers FJ, et al. Improvements in respiratory and psychosocial functioning following total laryngectomy by the use of a heat and moisture exchanger. *Annals of Otolaryngology, Rhinology & Laryngology*. 1993;102(11):878–883.

2. Ackerstaff AH, Hilgers FJ, et al. Heat and moisture exchangers as a treatment option in the post-operative rehabilitation of laryngectomized patients. *Clinical Otolaryngology*. 1995;20:504–509.


ADHESIVE HOUSINGS

InHealth Technologies® presents the FitSeries—adhesive housings designed to meet your skin and stoma needs after a total laryngectomy. Available in two styles, three shapes each. Compatible with Day&Night and SpeakFree Hands-Free HME cartridges.

Blom-Singer® HydroFit® Adhesive Housings

The Blom-Singer HydroFit Adhesive Housing is a hydrocolloid adhesive baseplate made with a protective gel-like layer and a moisture-absorbent adhesive. The HydroFit is gentle on the skin and flexible, but maintains strong adhesion. Available in three shapes: round, oval, and oval extra.

Ref	Description
BE 6085	Hydrofit Adhesive Housing, Round, pack of 30
BE 6086	Hydrofit Adhesive Housing, Oval, pack of 30
BE 6087	Hydrofit Adhesive Housing, Oval Extra, pack of 30

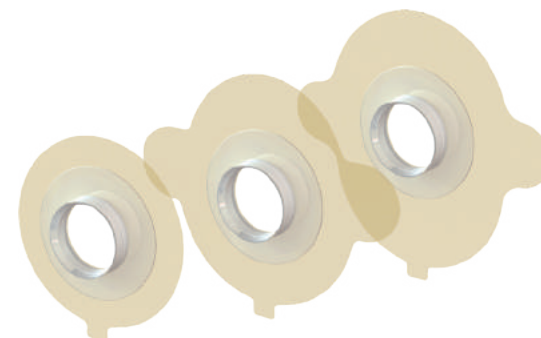

Blom-Singer® AccuFit® Adhesive Housings

The Blom-Singer® AccuFit® Adhesive Housing is designed to be our strongest adhesive option while still maintaining flexibility. It has increased elasticity and is less prone to tearing. Available in three shapes: round, oval, and oval extra.

Ref	Description
BE 6082	AccuFit Adhesive Housing, Round, pack of 30
BE 6083	AccuFit Adhesive Housing, Oval, pack of 30
BE 6084	AccuFit Adhesive Housing, Oval Extra, pack of 30



INTRODUCING

Blom-Singer® TruFit™ Adhesive Housing

The Blom-Singer® TruFit™ Adhesive Housing offers a robust baseplate compatible with deeper tracheostomas. Engineered with improved support to facilitate hands-free speech when used with a Blom-Singer® voice prosthesis and the SpeakFree® HME.

Ref	Description
BE 6088 *	TruFit™ Adhesive Housing, Oval Extra, pack of 30

*Awaiting launch on Prescription



Oval Extra

TRACHEOSTOMA ATTACHMENTS

Blom-Singer® Tracheostoma Valve Housings

The Tracheostoma Valve Housing is a reusable, flexible housing that attaches to the Day&Night and SpeakFree Hands-Free HME cartridge or Shower Guard over the stoma. It is applied with double sided tape discs. It is available in two sizes, standard and large.

Ref	Description
BE 6038	Tracheostoma Valve Housing, standard, 1 ea
BE 6039	Tracheostoma Valve Housing, large, 1 ea



Blom-Singer® Adhesive Tape Discs

A variety of double-sided tape discs are available to provide attachment of the tracheostoma valve housing.

Ref	Description	Ref	Description
BE 6041	Regular Tape Discs, standard, pack of 30	BE 6044	Thin Tape Discs, large, pack of 30
BE 6042	Regular Tape Discs, large, pack of 30	BE 6034	Heavy Duty Tape Discs, standard, pack of 30
BE 6043	Thin Tape Discs, standard, pack of 30	BE 6035	Heavy Duty Tape Discs, large, pack of 30



● AND BEYOND

SKIN CARE

Remove™ Adhesive Remover

Gently cleans hydrocolloid, acrylic and rubber-based adhesive residues from the skin without irritation, discomfort or trauma.

Ref	Description
59403125	Adhesive Remover, box of 50



Blom-Singer® Shower Guard

The Blom-Singer Shower Guard is a one-piece, right-angle plastic shower guard that comes with a Tracheostoma Standard Adhesive Housing and 30 double-sided Adhesive Tape Discs. Also compatible with the Blom-Singer StomaSoft® Laryngectomy Tube.

Ref	Description
BE 6048	Shower Guard—Tracheostoma Valve Housing (BE 6038) and Regular Tape Discs, pk 30 (BE 6041) included



Introducing:



A prescription delivery service, from a team with decades of experience in working with people living with a neck stoma. Our solution to the issue of the avoidable delays seen in the supply of tracheostoma products.



SevernDIRECT: We Understand **You** and **Your** Needs

- **Your** products; no changes
- A solution to speed up supply of **your** products
- Ample stocks of all prescription items held by **SevernDIRECT**
- Delivered directly to **your** door; no quibble
- No delays caused by a middleman
- Making **your** life easier

SevernDIRECT offer a free, unbiased prescription items delivery service, dedicated to people who have undergone Laryngectomy and Tracheostomy surgery.

The Benefits for **You** include:

- ✓ A customer care team who have a deep understanding of **your** needs, a person living with a neck stoma
- ✓ A prompt supply of laryngectomy or tracheostomy products from all manufacturers, as prescribed by **your** clinician, directly to **your** door
- ✓ A promise to contact **you** only when you've requested or when it's necessary, about supply of laryngectomy or tracheostomy products as prescribed by **your** clinician

This service launched March 2024

If **you** like the sound of **SevernDIRECT** & would like to use this service and/or move away from your current provider, then please contact us:

✉ Email: customercare@severndirect.com

☎ Phone: 0800 160 1844

✉ Post: **SevernDIRECT**, c/o Rapidcare, Koppa House
Napier Way, Crawley, RH10 9RA, United Kingdom



TO LEARN MORE

Scan the QR code to visit
our **SevernDIRECT** webpage



Professional Products

VOICE PROSTHESIS MANAGEMENT

Blom-Singer® Voice Prosthesis Sizers

The Blom-Singer Voice Prosthesis Sizer is a device used by clinicians to measure the length of the tracheoesophageal puncture to assist in selection of the appropriate voice prosthesis.

Ref	Description
BE 6036	Voice Prosthesis Sizer with 3 inserters (for 16 Fr voice prostheses)
BE 2036	Voice Prosthesis Sizer with 3 inserters (for 20 Fr voice prostheses)
BE 6096	Replacement Inserters (for use with both BE 6036 and BE 2036 Sizers), 10 ea

Blom-Singer® Tracheoesophageal Puncture Dilators

Ref	Description	Rx
BE 6050	18 Fr Tracheoesophageal Puncture Dilator (for 16 Fr prostheses)	■
BE 2050	22 Fr Tracheoesophageal Puncture Dilator (for 20 Fr prostheses)	■

Blom-Singer® Dilator-Sizers

The Blom-Singer Dilator-Sizer combines two tasks into one innovative product: sizing and dilation. In one step, the clinician can dilate and measure the tracheoesophageal puncture during voice prosthesis placement procedures.

Ref	Description
BE 2051	18 Fr Dilator-Sizer (for 16 Fr prostheses)
BE 2052	22 Fr Dilator-Sizer (for 20 Fr prostheses)

Blom-Singer® Replacement Gel Caps

Ref	Description	Rx
BE 3190	Replacement Gel Caps, 16 Fr, 90 ea	■
BE 3195	Replacement Gel Caps, 18 Fr, 90 ea	■
BE 3290	Replacement Gel Caps, 20 Fr, 90 ea	■
BE 3295	Replacement Gel Caps, 22 Fr, 60 ea	■



Manometer with Manometer Adaptor

A tool to teach patients how to self-regulate intratracheal air pressure. Proper airflow control may allow for better voice quality, less fatigue, and longer duration of the tracheostoma adhesive housing seal.

Ref	Description
BE 8000	Manometer Kit includes 1 manometer, 1 manometer adaptor and 1 adhesive housing
BE 8020	Replacement Manometer Adaptor includes 5 manometer adaptors and 5 adhesive housings



Blom-Singer® Insufflation Test Set

The Blom-Singer Insufflation Test Set is designed to aid the clinician in subjectively predicting the ability of the laryngectomized patient to produce tracheoesophageal voice.

Ref	Description
BE 6040	Insufflation Test Set, box of 6



Katz Extractor[®]

OTO-RHINO FOREIGN BODY REMOVER

The easy way out



Katz Extractor® Oto-Rhino Foreign Body Remover


Designed in conjunction with David L. Katz, MD, MPH, the Katz Extractor Oto-Rhino Foreign Body Remover is intended for extraction of impacted foreign bodies in the nasal passage. Single use only. This device may also be used in the external ear canal, based upon clinical judgment. Covered by one or more of the following patents and other patents pending: US#5,454,817. Katz Extractor® is a registered trademark of Freudenberg Medical, LLC.

Ref	Description
FB 5000	Oto-Rhino Foreign Body Remover, box of 5

TO LEARN MORE

Scan QR code
to visit the
Katz Extractor website



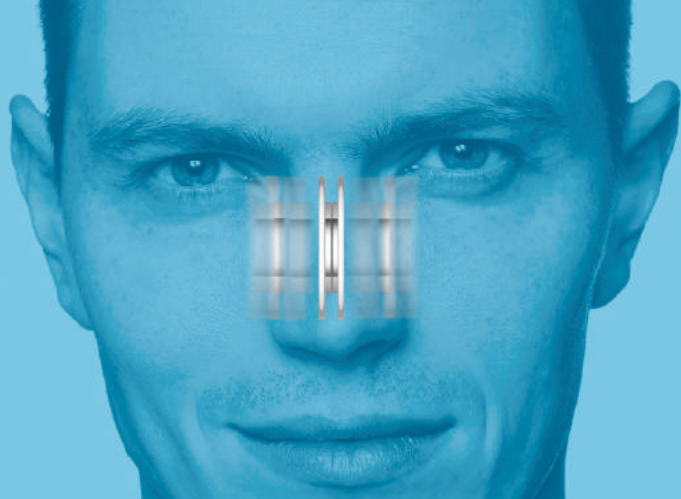

Nasal Septal Perforation Prosthesis

A two-piece, magnetically coupled solution for non-surgical closure of nasal septal perforations.

Nasal Septal Perforation Prostheses

The Blom-Singer® Nasal Septal Perforation Prosthesis is a two-piece, magnetically coupled solution for non-surgical closure of nasal septal perforations.







OVAL

Ref	Description
SP 2112	Nasal Septal Perforation Prosthesis Oval 11 x 13 x 2mm
SP 2113	Nasal Septal Perforation Prosthesis Oval 11 x 13 x 3mm
SP 2132	Nasal Septal Perforation Prosthesis Oval 13 x 17 x 2mm
SP 2133	Nasal Septal Perforation Prosthesis Oval 13 x 17 x 3mm
SP 2152	Nasal Septal Perforation Prosthesis Oval 15 x 21 x 2mm
SP 2153	Nasal Septal Perforation Prosthesis Oval 15 x 21 x 3mm



ROUND

Ref	Description
SP 1052	Nasal Septal Perforation Prosthesis Round 5 x 2mm
SP 1053	Nasal Septal Perforation Prosthesis Round 5 x 3mm
SP 1072	Nasal Septal Perforation Prosthesis Round 7 x 2mm
SP 1073	Nasal Septal Perforation Prosthesis Round 7 x 3mm
SP 1102	Nasal Septal Perforation Prosthesis Round 10 x 2mm
SP 1103	Nasal Septal Perforation Prosthesis Round 10 x 3mm

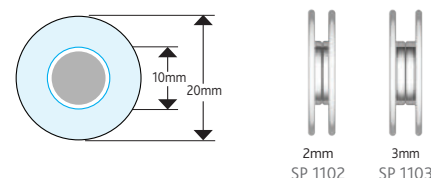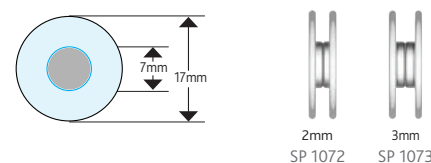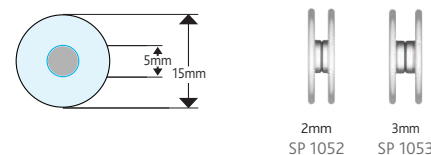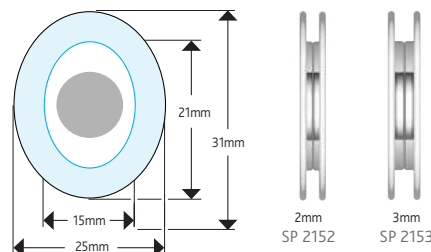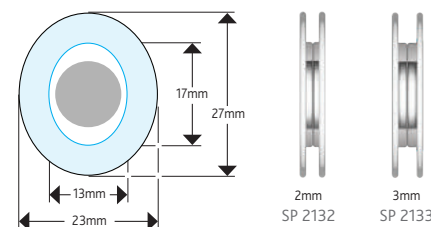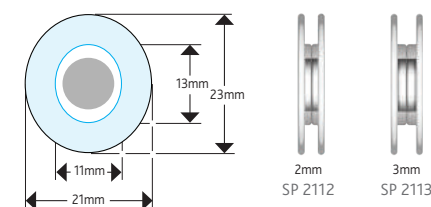

TO LEARN MORE
Scan QR code to visit the
Nasal Septal Perforation Prostheses website

NOTES:

This image shows a single sheet of white paper with horizontal ruling lines. The lines are evenly spaced and run across the width of the page. There are no margins, text, or other markings on the paper.

In 1978, Drs. Singer and Blom pioneered the methods of tracheoesophageal puncture and valved silicone voice prostheses that have been the international standard for voice restoration for over 45 years, allowing thousands of patients worldwide to regain their ability to speak. As a leading supplier of advanced voice restoration systems, our goal at InHealth Technologies® is to help laryngectomees regain and preserve a vital part of their lives—their voice. We are proud to continue the Blom-Singer® legacy today with clinical support, input, and leadership.

Blom-Singer®





Janice
Year of TL: 2014
Wearing AccuFit
housing with
SpeakFree HME
Hands-Free Valve



SEVERN HEALTHCARE TECHNOLOGIES LTD

42 Kingfisher Court, Hambridge Road, Newbury, Berkshire, RG14 5SJ

Tel: 01635 887640

Email: service@severnhealthcare.com

www.severnhealthcare.com

INHEALTH
TECHNOLOGIES®

United States
1110 Mark Ave.
Carpinteria, CA 93013

International
Tel: +1.805.684.9337
Fax: +1.805.684.8594
E-mail: international@inhealth.com

Terms & Conditions: https://inhealth.com/media/documents/pdf/Terms_and_Conditions.pdf
©2024 InHealth Technologies®—Manufactured by Freudenberg Medical, LLC (230911.01)

Blom-Singer®

The voice of experience since 1978



Order online at www.inhealth.com